

Pinch Calibration

Zero Calibrate

Place the pinch dynamometer flat on a table as pictured below. Make sure there is no externally applied weight on the gauge, then click the "Zero " button in the Pinch Gauge Calibration dialog.



Zero Calibrate

Weight Calibrate

Place the weight supplied with the system on the dynamometer as pictured below. Ensure that the grooves in the weight line up with the edges of the pinch dynamometer and the weight is clear of the pinch gauge cover. Type the weight of the object into the dialog box and click on the button labeled "Weight Calibrate."



Weight Calibrate